(FR)

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(EN)

**Presentacion**

Endosolv is an eugenate-based and resin-based root canal filling dissolving agent when an endodontic retreatment is required. 13 ml bottle.

**Composition**

Ethyl acetate, Amylacetat, Thymol.

**Intended use**

Solvent for root canal sealers.

**Target group(s)**

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Mature and permanent teeth.

**Intended users**

Dental healthcare professional.

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Preservation of tooth functionality.

**Performance characteristics**

**Clinical performance**

Dissolves or softens zinc oxide eugenol based or phenolic resin based material used in endodontic treatments.

**Indications for use**

Root canal sealers solvent when canal retreatment is required.

**Contra indication**

Hypersensitivity to one of components.

**Instructions for use**

The operative field shall be completely isolated with a rubber dam around the tooth.

- Mechanically clean out the pulp chamber and the entrance to the canals.
- Place several drops of Endosolv in an adapted content.
- Place a drop of Endosolv into the chamber.
- Clean the canal, using first a probe, then a broach, always dipping the point of the instrument in Endosolv before inserting it in the canal. The instrument must be removed from the canal and wiped as soon as resistance is felt.
- Repeat the operation several times, until reaching the apex. Intermediate irrigation with sodium hypochlorite is effective in neutralizing the medium.
- Close the bottle after use.
- Any remaining product that was sampled from the bottle shall be discarded.

**Warnings and precautions for use**

**Precautions for use**

After using Endosolv to remove the old filling material, it is essential to prepare the tooth again and to carefully dry it, particularly when using a zinc-oxide eugenol cement as the new filling material. Handle the bottle carefully.

Keep the product in its original packaging.

May cause drowsiness or dizziness.

**Warnings**

Highly flammable liquid and vapour.

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Hypersensitivity reaction.

Please report any serious incident that occurred in relation to the device to Septodont (vigilance@septodont.com) and to the local competent authority for medical devices.

**Storage**

Store between 5°C and 25°C.

Use the device within 6 months after first opening.

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