# Q SOLUCIÓN EDTA

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(EN)

### INDICATIONS / PROPERTIES:

• The ethylene diamine tetra acetic acid and its derivatives have a strong chelating power over various ions. This property gives them many applications. In Odontology they are widely used as dentin softeners to make easier the dentin remove. • In core preparations Q Solution remove easily the smear layer. • In canal instrumentation the irrigation of Q Solution produces a soften effect in the dentin that helps the instrumentation, the dentin gets softer making easier and faster the procedure

### INSTRUCTIONS FOR USE:

• Irrigate with a syringe inside the canal. • Leave approx 30 seconds and continue with the instrumentation in the usual way.

### PRECAUTIONS:

• A rubber dam is recommended to isolate the area • Do not leave rest of Q Solution inside the canal and avoid the product pass through the apex as it can produce discomfort. Even if it happens no toxic effects should appear. • After the instrumentation the canal must be rinsed and dried completely with paper points. • Caution to allergic reactions to EDTA, benzalkonium chloride or NaOH. • Keep the container tightly closed.

### COMPOSITION:

• EDTA (Na<sub>2</sub>) 17%. • Benzalkonium chloride 8%. • Neutralized Excipients.

### STORAGE:

Date of caducity and lot in the bottle and base of box.

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