Prime&Bond active™

Universal Adhesive

Caution, consult
instructions for use

20 sec. Agitate slightly	5 sec. Remove solvent	Cure ≥ 500 mW/cm ² : 20 sec. ≥ 800 mW/cm ² : 10 sec.

Dentsply DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
www.dentsplysirona.com



© 2017 Dentsply Sirona, Inc.

THE DENTAL
SOLUTIONS
COMPANY™

 **Dentsply
Sirona**

Step-by-step instructions: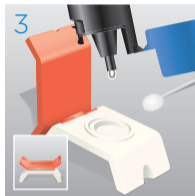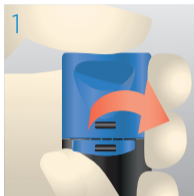

Preparation

Pull off originality seal.

Flip Top Bottle can be opened and closed with one hand. Do not screw off the cap.

Dispense Prime&Bond active™ into a CliXdish™. In a closed CliXdish™ Prime&Bond active™ remains usable for up to 30 minutes.

To close the bottle, press down the cap until it locks in place with a perceptible “click”.

Cleaning & Disinfection
Disassemble CliXdish™ for reprocessing.

OR

Single Unit Dose

Grasp container at each end, placing thumb along center score. Firmly apply pressure until container separates.

Insert applicator into opening to saturate applicator tip.

OPTIONAL

Etching

Start etching at enamel margins.

Extend to dentin.


Rinsing

Remove acid gel and rinse.


Remove excess water.
Do not desiccate!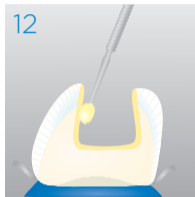


Application

Apply Prime&Bond active™ to all cavity surfaces.
Avoid pooling.

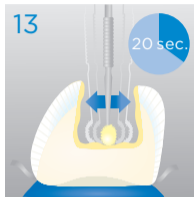
Keep Prime&Bond active™ slightly agitated for 20 seconds.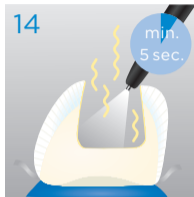

Evaporation

Evaporate solvent with air for at least 5 seconds.

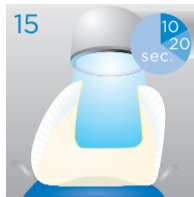Light Curing

$\geq 500 \text{ mW/cm}^2 = 20 \text{ s}$
 $\geq 800 \text{ mW/cm}^2 = 10 \text{ s}$

Prime&Bond active™

Universal Adhesive

Universaladhäsiv

Adhésif Universel

Adesivo Universale

Adhesivo Universal

Adesivo universal

Universal Adhesiv

Universeel adhesief

Universal adhæsiv

Universal Adhesiv

Universal sidosaine

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Universalusis rišiklis

Instructions for Use	English	2
Gebrauchsanweisung	Deutsch	12
Mode d'emploi	Français	22
Istruzioni per l'uso	Italiano	32
Instrucciones de uso	Español	42
Instruções de utilização	Português	52
Bruksanvisning	Svenska	62
Gebruiksaanwijzing	Nederlands	72
Brugsanvisning	Dansk	82
Bruksanvisning	Norsk	91
Käyttöopas	Suomi	101
Lietošanas instrukcija	Latviešu	110
Naudojimo instrukcijos	Lietuvių k.	119

Prime&Bond active™

Universal Adhesive

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Placement of direct light cured restorations: Step-by-step	5
4 Repair of composite, ceramic and amalgam restorations with light cured composites: Step-by-step	7
5 Cavity varnish for use with fresh amalgam: Step-by-step	7
6 Cementation of indirect restorations using Calibra® Ceram: Step-by-step	8
7 Cementation of indirect restorations with other dual cure and self cure resin cements Step-by-step	8
8 Endodontic post cementation	10
9 Hygiene	10
10 Lot number and expiration date	11

1 Product description

Prime&Bond active™ universal adhesive is a combined Etch&Rinse (Total Etch), Self Etch and Selective Etch adhesive. It offers a simple adhesive application technique for both direct and indirect indications and bonds to enamel, dentin, composites, zirconia and metals.

1.1 Indications

- Direct, light cured composite and compomer restorations.
- Light cured resin cemented veneers.
- Composite, ceramic and amalgam repairs.
- Cavity varnish for use with fresh amalgam.
- Indirect restorations and endodontic posts cemented with Calibra® Ceram.

In combination with Dentsply Sirona Self Cure Activator:

- Direct dual cure/self cure composite restorations and core build-ups.
- Cementation of indirect restorations and endodontic posts using dual cure/self cure resin cements.

1.2 Contraindications

- Use with patients who have a history of severe allergic reaction to (meth)acrylate resins or any of the components.
- Direct application to dental pulp (direct pulp capping).

1.3 Delivery forms¹

Prime&Bond active™ universal adhesive is available in:

- Tri-layer bottle with flip-top-cap
- Disposable containers for single patient use

1.4 Composition

- Phosphoric acid modified acrylate resin
- Multifunctional acrylate
- Bifunctional acrylate
- Acidic acrylate
- Isopropanol
- Water
- Initiator
- Stabilizer

1.5 Compatible materials

Prime&Bond active™ universal adhesive is compatible with conventional (meth)acrylate-based visible light cured restorative and cement materials.

Prime&Bond active™ universal adhesive is compatible with Dentsply Sirona Calibra® Ceram and Calibra® Veneer (see respective Instructions for Use for detailed instructions).

When used with Dentsply Sirona Self Cure Activator Prime&Bond active™ universal adhesive is compatible with dual cure/self cure resin cements, composites and core build-up materials.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains polymerizable (meth)acrylate monomers which may be irritating to skin, eyes and oral mucosa. The material does not contain HEMA but may cause allergic contact dermatitis in susceptible persons. The material is acidic and may cause burns.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, redish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

¹ Some delivery forms may not be available in all countries.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
- Use in a well-ventilated area. Avoid inhaling vapor.
- Flammable: Keep away from sources of ignition. Take precautionary measures against static discharges.
- Prime&Bond active™ universal adhesive is a light cured material. Protect from ambient light.
- Applicator Tips are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Unit Dose container is intended for single use only. Discard Single Unit Dose after use. Do not reuse for other patients in order to prevent cross-contamination.
- Unprepared enamel must be etched using phosphoric acid before application of Prime&Bond active™ universal adhesive.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close bottle immediately after use.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening and cause softening of the product.
 - Avoid the product saturating the gingival retraction cord. If it soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
 - If mineral-impregnated (e.g. ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Keep the product out of direct sunlight and store in a well-ventilated place.
- Store at temperatures from 2 °C to 24 °C (35 °F-75 °F). Use the product at room temperature.
- Do not use after expiration date.

3 Placement of direct light cured restorations: Step-by-step

3.1 Isolation, cleaning and conditioning of enamel and dentin

1. Use adequate isolation such as rubber dam.
2. Clean unprepared enamel and dentin with a rubber cup and pumice or a cleaning paste such as Nupro® Prophylaxis Paste.
3. Wash prepared and unprepared enamel and dentin thoroughly with water spray and blot or lightly air dry. Do not desiccate dentin.

3.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp with a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see Instructions for Use) leaving the rest of the cavity surface free for bonding.

3.3 Acid conditioning of enamel and dentin

Prime&Bond active™ universal adhesive offers operators the choice of enamel and dentin pre-treatment:

➔ Self etch mode

1. No etching with phosphoric acid, proceed directly to application of the adhesive.

➔ Selective enamel-etch mode

1. Apply phosphoric acid (e.g. Conditioner 36) according to manufacturer's instructions.
2. Gently extrude conditioner gel to the enamel (margins) only.
3. For best results, condition **enamel for at least 15 seconds**. Any incidental contact of conditioner gel to dentin should be limited to 15 seconds or less.
4. Remove gel with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
5. Remove rinsing water completely by blowing gently with an air syringe or blot dry.
6. Do not desiccate dentin.
7. Proceed immediately to application of the adhesive.

➔ Etch-and-rinse mode

1. Apply phosphoric acid (e.g. Conditioner 36) according to manufacturer's instructions.
2. Gently extrude conditioner gel to the cavity surfaces starting at the enamel margins.
3. For best results, condition **enamel for at least 15 seconds** and **dentin for 15 seconds or less**.
4. Remove gel with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
5. Remove rinsing water completely by blowing gently with an air syringe or blot dry.
6. Do not desiccate dentin.
7. Proceed immediately to application of the adhesive.

Contamination.

Restoration failure.

1. Once the surfaces have been properly treated, they must be kept uncontaminated.
2. If salivary contamination occurs after acid conditioning, thoroughly clean with vigorous water spray, dry and apply adhesive as described in chapters 3.4 to 3.6 without repeating the acid conditioning step.

3.4 Dosage

Unit Dose Container:

1. Grasp container at each end, placing thumb along center score.
2. Firmly apply pressure and twist the cap until container separates.
3. Discard separated cap. Unit Dose container may be placed on the tabletop or held between fingers for convenience.
4. Insert disposable applicator into opening to saturate Applicator Tip.

Flip-top Bottle:

1. Pull off originality seal.
2. To open, take the bottle in the hand, place a thumb in the thumb recess and push the cap to the side until it remains in the open position.
3. Hold the bottle upside-down in a vertical position and dispense 1-2 drops of adhesive into a CliXdish™ mixing well or standard dappen dish or mixing well.
4. If necessary clean dropper of the bottle using a soft paper tissue.
5. Carefully close the bottle again by applying pressure to the lever from above. The cap snaps into place when pressed closed with a perceptible “click”.

Technique Tip: CliXdish™.

Material in a closed CliXdish™ mixing well will remain usable for **up to 30 minutes**.
Material dispensed into a standard mixing well must be used immediately.

3.5 Application of adhesive to enamel and dentin

1. Use a new Applicator Tip to apply adhesive to completely wet the surfaces to be treated. If necessary rewet Applicator Tip. Avoid pooling of the adhesive.
2. Keep the adhesive slightly agitated for 20 seconds.
3. Disperse adhesive and remove solvent with clean, dry air from an air-water syringe. Treat every surface with a **moderate air flow** for at least 5 seconds until a glossy and uniform layer results. Avoid dry spots and splashing of adhesive by using a too strong air flow.

Insufficient evaporation of solvent.

Inadequate polymerization.

1. Follow strictly the instructions of the steps above.

3.6 Light curing of adhesive

1. Light cure adhesive with a suitable curing light². Cure adhesive for **10 seconds** when using a curing light as Dentsply Sirona SmartLite® PS or SmartLite® Focus® that has a minimum irradiance of 800 mW/cm².
Cure adhesive for **20 seconds** when minimum irradiance is between 500 mW/cm² and 800 mW/cm².
Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.

² Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing time.
3. Check minimum irradiance.

2. Proceed immediately to placement of the direct restoration or the preparation and cementation of indirect restoration as described in the following chapters.

3.7 Placement of direct restorations with light cured composites and compomers

1. After application and light curing of adhesive place restorative following the Instructions for Use of the used composite or compomer material.
2. Ensure that the cured adhesive surfaces remain uncontaminated before applying the direct restorative material. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry and reapply adhesive as described in chapters 3.4 to 3.6 without repeating the acid conditioning step.

4 Repair of composite, ceramic and amalgam restorations with light cured composites: Step-by-step

In case of composite and amalgam repair:

1. Create mechanical retention where possible. Roughen the fractured surface for best results, sandblast the area to be repaired with an intraoral microetcher (50 µm alumina). Rubber dam is recommended with high speed evacuation.
2. Rinse with water for 10 seconds (microetched areas for 15-20 seconds), air dry, and apply adhesive [see chapters 3.4 to 3.6].
3. Treat enamel and dentin surfaces according to the procedure described in chapters 3.3 to 3.6.
4. Proceed immediately to completion/restorative material placement following the Instructions for Use of the used composite material.

In case of glass ceramics repair:

1. Etch with buffered hydrofluoric acid for intraoral use according to manufacturer's instructions and apply silane (e.g. Calibra® Silane Coupling Agent) to ceramic surfaces to be repaired following manufacturer's instructions.
2. Treat enamel and dentin surfaces according to the procedure described in chapters 3.3 to 3.6.
3. Proceed immediately to completion/restorative material placement following the Instructions for Use of the used composite material.

5 Cavity varnish for use with fresh amalgam: Step-by-step

1. Apply adhesive to cavity surfaces without acid conditioning as described in chapters 3.4 to 3.6.
2. Proceed immediately to completion/restorative material placement.

6 Cementation of indirect restorations using Calibra® Ceram: Step-by-step

Calibra® Ceram.

When using Calibra® Ceram the Dentsply Sirona Self Cure Activator is **not** required.

Light curing.

Light curing of Prime&Bond active™ may be accomplished right after seating the restoration with Calibra® Ceram in case of light transmissible indirect restorations (≤ 2.5 mm thick ceramic and composite) only.

Application of adhesive.

Application of adhesive to the inner surface of glass ceramic restorations treated with silane is not required.

1. Apply adhesive to prepared tooth surfaces according to the procedure described in chapters 3.4 to 3.6.
2. Treat internal surface of restoration according to manufacturer's or dental laboratory's instructions, i.e., etching or mechanical roughening.
3. In case of glass ceramics etch with hydrofluoric acid according to the manufacturer's instructions and apply silane (e.g. Calibra® Silane Coupling Agent) following manufacturer's instructions. In case of zirconium dioxide or metal apply Prime&Bond active™ universal adhesive to inner surfaces of restorations to enhance adhesion.
4. Proceed immediately to cementation of indirect restoration following the Instructions for Use for Calibra® Ceram.

7 Cementation of indirect restorations with other dual cure and self cure resin cements: Step-by-step

Dentsply Sirona Self Cure Activator.

Dentsply Sirona Self Cure Activator is required. The Self Cure Activator (SCA) is used with Prime&Bond active™ to be compatible with conventional methacrylate-based dual and self cured composite restorative materials including Dentsply Sirona dual and self cured composite restorative and cement materials.

Application of adhesive.

Application of adhesive to the inner surface of glass ceramic restorations treated with silane is not required.

7.1 Pulp protection and tooth conditioning

For pulp protection and tooth conditioning, please see chapters 3.1 to 3.3.

7.2 Dosage and mixing

1. Dispense 1-2 drops of Prime&Bond active™ into a clean CliXdish™ Mixing Well or standard plastic mixing well, followed by Dentsply Sirona Self Cure Activator (SCA), at a ratio of 1:1 (same number of drops).
2. Mix contents for 1-2 seconds with a new Applicator Tip.

7.3 Application of mixed adhesive/activator to enamel and dentin

1. Use the Applicator Tip to apply mixed adhesive/activator to completely wet the surfaces to be treated. If necessary re-wet Applicator Tip. Avoid pooling of the adhesive/activator.
2. Keep the mixed adhesive/activator slightly agitated for 20 seconds.
3. Disperse mixed adhesive/activator and remove solvent with clean, dry air from an air-water syringe. Treat every surface with a **moderate air flow** for at least 5 seconds until a glossy and uniform layer results. Avoid dry spots and splashing of mixed adhesive/activator by using a too strong air flow.

Insufficient evaporation of solvent.

Inadequate polymerization.

1. Follow strictly the instructions of the steps above.

7.4 Light curing of mixed adhesive/activator

1. Light cure adhesive/activator with a suitable curing light³ for **10 seconds** when using a curing light as Dentsply Sirona SmartLite® PS or SmartLite® Focus® that has a minimum irradiance of 800 mW/cm². Cure adhesive/activator for **20 seconds** when minimum irradiance is between 500 mW/cm² and 800 mW/cm². Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing time.
3. Check minimum irradiance.

2. Proceed immediately to placement of the indirect restoration.

7.5 Preparation of indirect restorations

1. Treat internal surface of restoration according to manufacturer's or dental laboratory's instructions, i.e., etching or mechanical roughening.
2. In case of glass ceramics etch with hydrofluoric acid according to the manufacturer's instructions and apply silane (e.g. Calibra® Silane Coupling Agent) following manufacturer's instructions. In case of zirconium dioxide or metal apply mixed adhesive/activator to inner surfaces of restorations to enhance adhesion.
3. Prepare, mix and apply dual cure or self cure composite or dual cured resin cement, according to manufacturer's instructions.

³ Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm.

8 Endodontic post cementation

8.1 With core-X® flow core build-up material: Step-by-step

1. Post space must be rinsed with water and dried using suction and/or paper points to leave a moist surface.
Optional post space may be etch-and-rinse treated with phosphoric acid according to the procedure described in chapters 3.3.
2. Mix Prime&Bond active™ universal adhesive with Dentsply Sirona Self Cure Activator as described in chapter 7.2.
3. Apply mixture of Prime&Bond active™ universal adhesive and Dentsply Sirona Self Cure Activator to post space according to the procedure described in chapter 7.3. Use of paper points after application helps to avoid pooling of adhesive in post space.
Light curing of adhesive before post cementation is optional – follow chapter 7.4.
4. Treat post surface according to the post manufacturer's instructions (i.e. for X-Post® fibre post apply adhesive mixture and air dry).
5. Proceed immediately to endodontic post cementation following the Instructions for Use for core-X® flow.

8.2 With other dual cure and self cure resin cements: Step-by-step

1. Post space must be etch-and-rinse treated with phosphoric acid according to the procedure described in chapters 3.3.
2. Mix Prime&Bond active™ universal adhesive with Dentsply Sirona Self Cure Activator as described in chapter 7.2.
When using Calibra® Ceram the Dentsply Sirona Self Cure Activator is not required.
3. Apply mixture of Prime&Bond active™ universal adhesive and Dentsply Sirona Self Cure Activator to post space according to the procedure described in chapter 7.3. Use of paper points after application helps to avoid pooling of adhesive in post space.
4. Light cure adhesive before post cementation following chapter 7.4.
5. Treat post surface according to the post manufacturer's instructions.
6. Proceed immediately to endodontic post cementation following respective Instructions for Use.

9 Hygiene

Cross-contamination.

Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

9.1 CliXdish™ mixing well

For reprocessing instructions please refer to the Instructions for Use of the CliXdish™, which is available on our webpage at www.dentsplysirona.com and www.dentsply.eu/IFU.

If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. You can use the order form provided on our webpage for this purpose.

10 Lot number () and expiration date ()

1. Do not use after expiration date.
ISO standard is used: "YYYY-MM" or "YYYY-MM-DD".
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2017-10-24

[These Instructions for Use are based on Master Version 07]

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