Zeta 3 Wipes Pop-Up

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EN - INDICATION OF USE: Wipes designed to clean and disinfect non-invasive and non-submersible medical devices (small medical and dental devices).

INSTRUCTIONS FOR USE: Ready-to-use wipes. Single use wipes. When using the sachets the first time:

- Open the winder, permanently remove the tamper-proof seal.
- Thread the first wipe through the hole and then close the winder.
- Use the wipe on the equipment.
- Leave on for 30 seconds to 15 minutes according to the antibacterial activity required. Leave to dry in the open air. Do not rinse.
- Close the winder carefully after each use to prevent the wipes from drying out.

PRECAUTIONS: Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of contents/container to an approved waste disposal plant. Keep out of reach of children. Wear protective gloves. IF IN EYES: Rinse cautiously with water for several minutes. Never reuse wipes, they may be less effective and there is a risk of cross contamination. To be used within 3 months after opening.

CONTAINS IN PARTICULAR: Less than 5% non-ionic surfactants, perfumes (including limonene), disinfectants : Ethanol (CAS 64-17-5), Alkyl (C12-16) dimethylbenzylammonium chloride (CAS 68424-85-1), Propan-2-ol (CAS 67-63-0).

SPECTRUM OF ACTION

Effectiveness*		Effective period under clean conditions
General treatment		
Bactericidal	EN 13727**, EN 14561**, EN 14561: MRSA	5 min
Fungicidal	EN 14562 (Aspergillus fumigatus)	5 min
Yeasitidal	EN 13624, EN 14562 (Candida albicans)	5 min
Tuberculoicidal	EN 14348, EN 14563 (Mycobacterium terrae)	15 min
Virucidal	EN 14476**(HBV, HCV, Adenovirus, Coronavirus, Norovirus, VRS, H1N1, HSV1), EN 14476 (Polymavirus)	1 min
	EN 14476 (Rotavirus)	15 min
Additional uses		
Bactericidal	EN 1276	30 s
	EN 13697	5 min
Yeasitidal	EN 1650**, EN 13697 (Candida albicans)	5 min

Skin tolerance tested under dermatological control. Manufactured in compliance with the NF EN ISO 13485 quality management systems. Data sheet available on request. CE marking affixation year: 2003.
*Properties of the impregnation solution. **In dirty conditions.

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