#### Product description:

**Bifix SE** is a universal, dual-curing, self-adhesive, luting composite. It is used for luting indirect restorations made from ceramic, zirconium dioxide, composite and metal as well as for luting posts. Additional adhesives or conditioners are not required when using **Bifix SE**. **Bifix SE** can be used with any commercial coupling silane (follow manufacturer's instructions).

**Bifix SE** contains bifunctional methacrylate, acid methacrylate and inorganic fillers. The filler content is 70 % w/w.

**Bifix SE** is available in the following shades: Universal (U), White-opaque (WO) and Transparent (T).

The QuickMix syringe automatically mixes the material and prevents errors and bubbles. **Bifix SE** comes with tapered cannulae (mixing tips type 14) for easily accessible cavities and restorations as well as blunt-end cannulae (mixing tips type 15). Attaching the intraoral tip 1 to the blunt cannula helps access difficult to reach areas and post drillings.

#### Indications:

- Definitive luting of inlays, onlays, crowns and bridges (not Maryland bridges) made from ceramic, zirconium dioxide, composites and metal
- Definitive luting of metal, ceramic, zirconium dioxide and fibre-reinforced posts

#### Contraindications:

**Bifix SE** contains methacrylates, organic acids, benzoyl peroxide amines and BHT. **Bifix SE** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

#### Patient target group:

**Bifix SE** is suitable for use in all patients without any age or gender restrictions.

#### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

#### User:

**Bifix SE** should only be used by a professionally trained dental practitioner.

#### Use:

##### Pulp protection:

Areas in close proximity to the pulp should be covered with a hard-setting calcium hydroxide preparation to avoid pulp irritations.

##### 1. Removing the temporary and preparing the cavity

Remove the temporary and thoroughly clean the cavity or prepared core, e.g. using oil- and fluoride-free cleaning paste.

##### 2. Preparation

Check the fit and contact points of the definitive restoration. If silicone has been used for verification, thoroughly remove silicone residues afterwards. Checking the occlusion on glass-ceramic restorations should only be carried out after definitive luting to reduce the risk of fracture. Remove any residue from the surface of the tooth and restoration after completing the try-in.

Prepare the restoration according to the manufacturer's instructions and dry it carefully. Dry the surface of the tooth, preferably with either a foam pellet or gentle puffs of air, leaving the dentine slightly moist.

##### 3. Inserting the restoration

Bring material to room temperature before use.

Due to technical reasons, there may be slight differences in the filling levels of the syringe chambers prior to first use. Therefore remove the syringe cap and extract material until both chambers deliver uniformly. Then fit a mixing tip type 14 or 15. The notches on the syringe and the mixing tip must line up exactly. Turn the mixing tip 90° clockwise to lock the tip into place. If necessary an intraoral tip type 1 can be attached to the end of the mixing tip type 15. To guarantee homogeneous mixing of both pastes, please discard the first 2 mm of the extracted material. After that, apply the material immediately and directly onto the prepared contact surfaces.

Insert the restoration with the aim of leaving a small amount of excess material.

#### Curing behaviour:

Working time: 90 s (23 °C)  
Intraoral working time after insertion: 30 - 60 s  
Setting time: 4 min (37 °C)

An evaluation of the curing of **Bifix SE** on mixing pads or similar does not provide valid evidence. This is due to the fact that **Bifix SE**, as is typical for luting composites, does not set hard completely (or not at all, where layers are thin) when exposed to oxygen at room temperature. Completion of curing is guaranteed after insertion of the restoration, at mouth temperature and unaffected by ambient oxygen.

The attached mixing tip is intended for single use only. After use, store the QuickMix syringe tightly sealed. Check that material flows freely through the openings before reusing.

The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

##### 4. Removal of excess material

Remove excess material from difficult to reach areas first. Clean contact points and interproximal areas with dental floss.

Excess cement can be removed with a suitable instrument (scaler) either during the gel phase (ca. 60 - 90 s after insertion) or after brief light-curing (1 - 2 s). Larger amounts of excess material are easier to remove. The restoration must be held in position during the polymerisation phase.

As an alternative, any coarse pieces of excess luting composite can be removed with a foam pellet or a similar object immediately after insertion. Leave a small amount of excess material and cover it with a glycerine gel (air-block) or light-cure.

##### 5. Curing

Polymerisation time can be controlled with additional light-curing. Cure translucent restorations with light for 20 s on each side. Light-curing the luting gap for 10 s on each side is recommended for opaque restorations. The autopolymerisation of **Bifix SE** is completed 4 min after insertion.

#### 6. Finishing

Excess material can be worked immediately after polymerisation. Remove cured excess material with a finishing diamond or flexible polishing disc. Finish approximal areas with finishing/polishing strips.

#### Luting endodontic posts:

1. Prepare root canal for treatment. Prepare the endodontic post according to the manufacturer's instructions.
2. Disinfect the root canal, rinse with plenty of water – do not use hydrogen peroxide.
3. Remove the transport cap from the QuickMix syringe. Subsequently attach the type 15 blunt mixing tip and turn to lock. Attach the type 1 intraoral tip to the mixing tip.
4. Dry the root canal and dentine with paper points, leaving the dentine slightly moist.

5. Applying **Bifix SE** directly into the root canal is recommended (begin applying from apical direction, allowing the intraoral tip type 1 to be immersed in the luting composite to prevent formation of air bubbles during the application). Do not use a lentalo! The endodontic post can also be wetted with **Bifix SE**. Insert the endodontic post with a turning motion with the aim of leaving a small amount of excess cement. Remove the rough excess material and light-cure **Bifix SE** in the direction of the endodontic post to anchor it.

#### Warnings, precautionary measures:

- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Phenolic substances, especially products containing eugenol and thymol, interfere with the curing of filling composites. For this reason, the use of zinc oxide eugenol cements or other materials containing eugenol in combination with **Bifix SE** should be avoided.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

#### Constituents (in descending order according to content):

Barium aluminium borosilicate glass, GlyDMA, UDMA, methacrylate phosphoric acid ester, BisGMA, fumed silica, HPMA, BisEMA, initiators, stabilisers, pigments

#### Storage:

Storage at 2°C–8°C. Do not use after the expiry date.

#### Disposal:

Dispose of the product in accordance with local regulations.

#### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Bifix SE** must be reported to VOCO GmbH and the responsible authority.

#### Note:

The Summary of Safety and Clinical Performance of **Bifix SE** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Detailed information can also be found at [www.voco.dental](http://www.voco.dental).

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VC 60 CC1784 E1 0922 99 © by VOCO