SEPARATOR

ES

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Separator material surfaces of the same nature, avoiding adhesion. Separator can be applied to reproduce gingiva on models in association with Gingifast Elastic Zhermack and/or Gingifast Rigid Zhermack, and in all cases in which it is necessary to prevent adhesion between compatible all cases in which it is necessary to prevent adhesion between compatible materials. Separator can be applied on C-Silicones (Zetalabor, Titanium) and A-Silicone (Platinum). Separator also reduces the inhibitions of A-Silicone cross-linking due to contact with incompatible materials (for example, for polyether-based impression materials). For best performance, only concerning polyether-based material, it is recommended to wait at least 5 hours after casting the impression before applying. The polyether impressions should be accurately washed and dried with a gentle blow of air before applying Separator.

INSTRUCTIONS FOR USE

This product can be applied either directly onto the dry impression (direct technique) or onto silicone moulds (indirect technique)

- Make sure that the surfaces to be isolated are thoroughly cleaned and dried.
- Spread Separator using the brush, or using the vaporizing accessory included in the kit.
- Wait for Separator to dry completely, until the surface becomes opaque. Drying time can be shortened by using a gentle blow of air.

WARNINGS

Close the bottle well after use.

Possible product deposits or separation of components are to be considered normal as they do not jeopardise product quality. If this happens, shake well before use.

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