## IT - Elite Cord

## Foglietto illustrativo

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## Instructions for use

### Patient target group

For the treatment of women and men of all ages as well as children.

### User

For dental use only.

### Description

Knitted retraction threads made of 100% cotton.

### Indications

Aid for pushing aside and retracting the gingivae.

### Contraindications

None

### Clinical benefit

Aid for preparing an impression for the fabrication of crowns or bridges

Aid for deep fillings or cervical fillings.

### Performance characteristics of the product

Numerous tiny loops form an interlocking chain. This unusual design results in even, lasting pressure on the gum line. Due to its special knitted construction, Elite Cord ensures excellent tissue displacement and is better retained in the sottilo than conventional retraction threads. Elite Cord thread also offer excellent haemostatic properties, as they absorb significantly greater amounts of haemostatic agents than conventional threads.

- Size #2 and occasionally size #3 are suitable for molars.

- Size #1 is often suitable for premolars and upper teeth.

- Sizes #0 or #00 are often used for the lower front teeth.

- The thread #000 is used for preparations or cementing of veneers and for thin, tightly fitting sottilo tissue. This size is also used as the lower thread for the double-thread technique. The nature of the tissue and the desired size of the free space may make it necessary to use sizes other than those described above. If preferred or necessary, the knitted retraction thread can be soaked in a haemostatic solution. The knitted thread can also be impregnated with aluminium chloride, a phosphate solution or an epinephrine hydrochloride solution.

### Specific precautions

Due to the type of packaging used, the retraction threads are pressed. Please select one size larger than expected.

### Application

1. Select the appropriate size of retraction thread.

2. Use a thread stopper to insert the retraction thread into the sottilo.

3. After inserting the retraction thread, rinse the affected areas. An optimal displacement of the tissue will take place within 1-3 minutes.

4. Remove the retraction thread from the sottilo before taking an impression. Lightly spray with water to clean the preparation and test the effect of the haemostatic agent.

### Warnings / residual risks

None

### Storage

Dry and at room temperature.

### Minimum shelf life

3 years if stored properly. See outer packaging.

### Variants

The different sizes of retraction cords

**REF C61000, size #000**

- Preparation and cementation of veneers

- Restorative treatment for thin and brittle tissue

**REF C61001, size #00 / REF C61002, size #0**

- Lower front teeth

- For luting veneers close to gingival or subgingival areas

- Restorations in classes III, IV + V

- Second thread for "two-thread" technique

**REF C61030, size #1**

- Sizes 1 and 2 are particularly effective for tissue control and/or restraining if they are previously immersed in a haemostatic solution before or after crown preparations. Use a protective "pre-preparation thread" for front teeth

**REF C61040, size #2**

- Upper thread for "two-thread" technique

- Protective "pre-preparation thread" for anterior teeth

### Disposal

Dispose of properly in accordance with the local official regulations.

Non-contaminated packaging can be sent to a recycling facility.

Side effects, reactions or other undesired occurrences for the patient and/or user (even in cases of doubt) must be reported to the manufacturer promptly and in detail.

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