



ED18



ED18

Function: Used to determine the position of the root canal, remove the dentin, calcification and loose crown at the 1/3 crown of the root canal.

Features: The tip has 12 micro-washers, and it is of good cutting performance.

Back and forth motion of tip brings more efficient operation.

In the case of waterless operation, select the mode of "Work for 5~10 seconds and then stop for 2 seconds", and open water every 60 seconds for cooling and rinsing.

Note:

(1) Due to the strong cutting performance of the tip, in order to prevent the penetration of the medullary cavity, it is recommended to use with a root canal microscope.

