®

# BADER

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CE

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## REVIEW MODULE WITH ASPIRATION

CE

Thank you for purchasing a product from our Bader® brand. To ensure correct use, please read the following instruction manual carefully.

## MAIN CHARACTERISTICS

Our work cabin is designed to provide you with an optimal experience in your daily work. With sturdy steel construction and an epoxy paint finish, we guarantee durability and reliability with every use. Key features of this cabin include:

**Built-in Drawer:** Offering a convenient storage solution for your essential tools and equipment.

**Knee Select Micromotor Compatibility:** The cabin is ready for the installation of a micromotor via a pedal, allowing you precise control over your operations.

**Personalized Regulation:**Control pressure and aspiration easily with our built-in regulators, adapting them to your specific needs.

**Effective lighting:**Thanks to the included light switch, you will always have a clear, well-lit view of your work area.

**Versatile Protection Screen:** The protective screen opens completely for easy access and cleaning, thus improving the convenience of use.

**Integrated Window:**A window in the protective screen allows for easier cleaning and handling.

**Integrated Blower:**Keep your work environment clean and free of particles with a built-in blower.

**Front Suction System:**Our front suction mouth improves air flow, preventing the creation of eddies and ensuring efficient extraction.

**Cold Light LED Light:** Cold light LED lighting provides soft white light that does not generate heat, making your work more comfortable and precise.

## CHARACTERISTICS TECHNIQUES

Dimensions: 90 cm x 60 cm x 137 cm (height)  
Bench: 30 mm special pressed laminate  
Construction: Powder-coated metal body and drawers

Metal covers: Electrostatically coated Metal profile leg: Yes  
Handles: Special chrome Pressed cylindrical rail: Yes  
Vacuum system: Fan motor and special carbon filter  
Environment ventilation: Contributes to ventilation  
Air injection system: With pedal Fixed  
lamp: On the bench  
Micromotor connection: Yes, on the table  
Folding cover: Plexiglas  
Eye Shield: Replaceable Glass Air  
Regulator: Yes

## SETTING UP FUNCTIONING

Start up:

Electrical Connection: To start the equipment, use the supplied cable to plug it into the electrical current using a conventional power plug. Make sure the mains voltage is correct.

Compressor Connection: Connect the compressor, which is located on the back left side of the device. Adjust the necessary pressure using the pressure regulator located inside the display in the cabin.

Maintenance:

Regular Cleaning: Maintenance is minimal, but it is important to keep the work area and equipment clean. Avoid excessive buildup of debris and ensure there are no blockages in the vacuum system.

Suction Bag: The suction bag is a key component. Periodically empty the suction bag, as this is where waste and absorbed particles accumulate.

By following these instructions, you will be able to start up your equipment and ensure that it operates efficiently and safely. Additionally, regular maintenance will ensure consistent performance over time.

## COMMERCIAL GUARANTEE CERTIFICATE

Dear customer: we express our gratitude for having placed your trust in a product of our brand whose quality has been verified under the strictest controls and is guaranteed by the present:

BADER EUROPE GROUP, SL with CIF B27804657 guarantees this product under the following terms and conditions:

- 1.The product covered by this certificate is protected for a period of ONE year from the date that appears on the receipt or invoice supporting the purchase, its territorial scope being all of Europe.
- 2.During said warranty period, in the event of a manufacturing defect, the consumer will have the right to have it repaired. Repairs will be free of charge and will be carried out within a reasonable period of time and without major inconvenience to the consumer, taking into account the nature of the product and the purpose it had for the consumer.
- 3.Failures, defects or damage caused to the product as a result of: mistreatment, improper use, exposure to adverse weather conditions, location in inappropriate environments for the product with dust, dirt and humidity, abuse, blows, are excluded from this guarantee. , falls, inadequate or insufficient installation, power supply failures, wear and tear, repairs carried out by people other than the indicated technical assistance, as well as alteration of the data contained in this certificate and/or other causes external to the certificate itself. product.
- 4.This guarantee will be automatically voided in the event of mishandling or if it has been used in violation of the rules contained in the manuals or instructions regarding use, operation, service and instructions for use.
- 5.The guarantee does not affect the rights available to the consumer in accordance with the provisions of the law.
- 6.In the event that any type of technical assistance related to the product is required, the consumer must contact the distributor or purchasing establishment.
- 7.For any issues that may arise due to the interpretation or execution of this guarantee, the provisions of the applicable national legislation will apply. 8. This certificate has been drawn up in accordance with the provisions of Law 23/2003 of July 10, on Guarantees in the sale of capital goods.
9. Enclose purchase receipt or invoice along with the duly completed warranty.

BRAND AND MODEL: ..... ESTABLISHMENT STAMP:

.....

SERIAL OR LOT NO: ..... DATE OF PURCHASE:

.....