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ENGLISH

Model

Silicone hose with revolving connection throughout 540° (Fig.1) for use with electric micromotors with light DEFINITIVE®LED or Bien Air MC3 LK.

Hose sizes

Diameter: 10.5 mm.
Standard length: 1.6 m.

Connections (Fig.2)

Blue tubing ①: air spray
Green tubing ②: water
Clear tubing ③: motor cooling air

Adjust pressures as indicated on the motor's operating instructions.
Maximum operating pressure of spray ducts and cooling air: 6 bar (87 psi).

Hose must be connected to a power supply unit meeting the requirements of CEI IEC 60601-1 Standard. Additionally, the power supply unit must have a minimum insulation strength of 500 V between the electric circuits and earth and a minimum insulation strength of 4000 V between the electric circuits and mains.

To connect electric wires, refer to the connection diagram of the motor's electronic control board.

Wirings for DEFINITIVE®LED micromotor

Black ④, red ⑦ and brown ⑥ electric wires (25 Vac or 60 Vdc max; 6 A max): supply of micromotor phases.
Blue electric wire ⑤ (3.5 V; 60 mA max): supply of the lighting device (LED).

Wirings for Bien Air MC3 LK micromotor.

Red ⑦ and Black ⑥ electric wires (25 Vac or 60 Vdc max; 6 A max): motor supply.
Brown ④ and blue ⑤ (negative pole) electric wires (3.5 V; 1 A max): bulb supply or LED.

Assembly

The point marked on the sticker put on the hose must be aligned to the direction of the bur on the contra-angle handpiece fit to the micromotor (Fig.3).

Keep the initial alignment of the wires and inner tubings.

Fix and slightly pull the locking string.

Remove the sticker when the hose has been installed.

Cleaning and disinfection

Clean with a cloth soaked in a disinfectant containing glutaraldehydes up to 3% or alcohols based on ethanol.

To keep the surface of the hose in good conditions, it is advisable to periodically wipe it with talcum powder.

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